**MAJOR
TOTAL
SOFT**

 **MAJOR**

MAJOR TOTAL SOFT

Permanently soft, polyvinyl siloxane liner for direct and indirect use.
Biocompatible, antibacterial, anallergic material. It doesn't release monomer and heat.

Major Total Soft can be used as a:

• Direct liner

Major Total Soft can be directly used in the dental cabinet to optimize denture stability and to obtain more accurate values.

• Therapeutical medium

Major Total Soft is used after extractions or implantological surgical operations in order to supply a soft and anallergic support to the patient and to make a quick recovery of the mucous membranes.

• Stabilizing medium

Major Total Soft can be used for compensating areas of mucous membranes which vary in resilience, sharp bony ridges or deep undercuts which may cause sufferings or denture instability. Major Total Soft, in fact, due to its excellent elasticity and softness, reduces masticatory loads, relieves pain and gives stability to the prosthesis.

• Temporary material

Major Total Soft can be used as a temporary material with every kind of total or partial prosthesis.

• Indirect liner

Major Total Soft can be processed in a dental laboratory, using standard reliners (Major Reliner) to rebase prosthesis.

• Repairing material


Both Major Total Soft base mass and its relevant Silicone Sealant can be employed in operations requiring to add or apply material on the prosthesis, even if the patient has already used the prosthesis for several days.

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Main Features:

- Major Total Soft has fluid consistency, it is easy to apply and it prevents air voids being entrapped during the dispensing of the material.
- Thank to its high resilience and final softness, Major Total Soft is well tolerated by patients.
- The silicone soft liner is a hydrophobic material and, combined with its relevant sealer (Silicone Sealant), it avoids liquids absorption and protects the prosthesis from infiltrations.
- Major Total Soft is monomer free.
- Major Total Soft is biologically inactive.
- The adhesive (MTS Adhesive) bonds the silicone soft liner to the denture acrylic reliably.
- Major Total Soft is smooth and dimensionally stable.
- Major Total Soft resists to the tensions exerted by masticatory loads.
- The silicone surface has no porosity and inhibits bacterial settling.
- Major Total Soft can be used directly by the dentist or it can be processed in a laboratory using standard reliners.
- The material is fully odourless and it doesn't lose colour.
- Major Total Soft is supplied in a pale rose colour suitable for every aesthetical need.

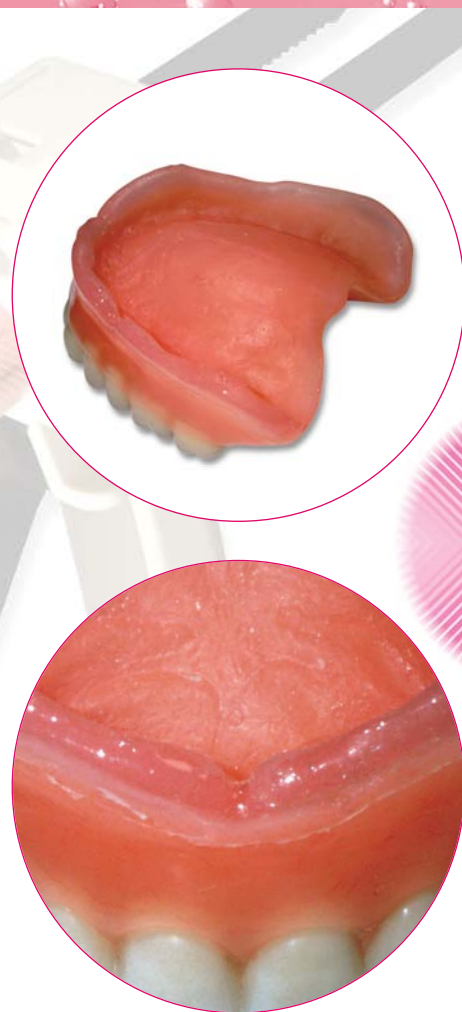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MAJOR Prodotti Dentari S.p.A.