- Prewash the printed parts in 97 % Isopropanol (IPA) in an ultrasonic bath for 2 minutes in a container, afterwards mainwash for additional 2 minutes in a container with fresh Isopropanol. Dry the printed part with compressed air. For a particular effective cleaning with isopropanol, the use of explosion-protected ultrasonic units is recommended.
- Post curing
PCU LED N₂ 12 min, 80 % light output (incl. nitrogen)
FotoDent® flash 12 min + 1 min gas pre flow
- Afterwards temper the denture base in an oven at 90 °C for 10 minutes.
- Recommended bonding adhesive for denture teeth: VITA VIONIC® BOND or IvoBase CAD Bond.
- In order to remove polymerized residues from the printer material container and to homogenise FotoDent® denture, remove the container from the printer and filter the FotoDent® denture through a fine 190 Micron paint strainer. To avoid bubbles, let the material rest for 1 to 2 hours prior to use, after filtration.

To avoid compromising material quality, do not expose the liquid material to light under any circumstances. Variations from the described manufacturing process may impair biocompatibility, lead to modified mechanical characteristics and/or color variations of the FotoDent® denture material.

Contra indication

FotoDent denture® is contraindicated for individuals with known allergy to methacrylates and acrylates.

Possible dangers

Before use observe the Material Safety Data Sheet of FotoDent® denture.

Hazard statements: **H315** Causes skin irritation. **H319** Causes serious eye irritation. **H317** May cause an allergic skin reaction. **H411** Toxic to aquatic life with long lasting effects.

Precautionary statements: **P261** Avoid breathing dust / fume / gas / mist / vapors / spray. **P273** Avoid release to the environment. **P280** Wear protective gloves / eye protection / face protection. **P305+P351+P338** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **P310** Immediately call a POISON CENTER / doctor. **P501** Dispose of contents / container in accordance with local / regional / national / international regulations.

Contains: Neopentylglycol(PO)2 Diacrylate, Diphenyl(2,4,6-trimethylbenzoyl)phosphinoxide, pigment paste

Lot number / Expiry date

The lot number and the expiry date are indicated on each FotoDent® denture packaging. In case of claims please always indicate the lot number of the product. Do not use the product after the expiry date.

Cleaning

For cleaning of FotoDent® denture dentures the use of MaxiFresh™ cleaning tablets is recommended. Please be aware of the MaxiFresh™ instruction for use. Long term contact of dentures with coloring beverages (e.g. tea or coffee) may discolor the denture.

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Intended use

FotoDent® denture is a methacrylate-based material for DLP systems with 385 nm-LED for production of denture base. Rapid Shape DII series and Asiga MAX UV are appropriate printers. For post curing of FotoDent® denture built parts, the Dreve light curing units PCU LED N₂ or FotoDent® flash are recommended for optimal results.

Indications for use

FotoDent® denture is a light curing resin intended for manufacturing of full and partial removable dentures.

Processing

- Optimal Orientation: vertical
- During the application of FotoDent® denture we recommend wearing personal protective equipment f. i. suitable gloves, safety goggles etc.
- FotoDent® denture should be well shaken for a minimum of 30 seconds, 1 hour before use.
- Carefully pour FotoDent® denture into the pre-determined container of the production unit.
- Please see further steps within printer manual.
- Remove all bubbles with a cleaned object.
- Processing temperature:

Rapid Shape DII series	23±3 °C
Asiga MAX UV	35±3 °C
- Select the product specific parameter settings for FotoDent® denture for your printing process.
- Direct finishing is recommended after completion of the printing process.
- After the platform has been raised, a dripping time of approx. 10 minutes is recommended.