

# Instructions for use RESIFLO

## INDICATION

RESIFLO is a ready-to-use light-curing product for modelling inlays, onlays, veneers, dental copings and for blocking out undercuts. RESIFLO is available in syringes

## INTENDED USES

- Modelling of inlays, onlays, veneers and crowns
- blocking out undercuts
- Fixing soldering points
- Correction and restoration of plaster models and dies
- Fixing orthodontics on the model

## APPLICATION DIRECTION

- Remove the cap from the RESIFLO, extrude enough material and recap the tube.
- RESIFLO can be applied with normal instruments with a working time of 3 min.
- RESIFLO can also be applied using neodyms directly on the model.
- Apply RESIFLO on the desired area to a maximum thickness of 3 mm and cure with a UV lamp or blue light source for 5 minutes.
- RESIFLO can be worked with using rotary tools after polymerization.
- RESIFLO can be polished to high gloss.
- RESIFLO burns without residue in ovens at 480°C / 900°F.
- RESIFLO polymerizes under the influence of daylight.

Do not remove resin from syringe if you do not intend to use it immediately.

## STORAGE

For optimum performance, we recommend keeping the syringe tightly closed, out of direct sunlight and at a temperature above 25°C / 77°F.